

# GlucoContro.online

## Connect with your patients' data remotely\*



To assist with diabetes management, invite your patients to share their blood glucose results.

[www.glucocontro.online](http://www.glucocontro.online)

\*Patients need to be registered in GlucoContro.online and approve sharing their data with their healthcare professional

# Easy access to patient information

Available on one screen with a single click



**Fast and easy access**  
to CONTOUR®DIABETES  
app data stored in  
CONTOUR®CLOUD



**Panel with general  
statistics and complete  
list of registered  
patients**



**Personalised information  
about each registered  
patient** (target ranges,  
time frames)



**Paul Smith** ✓

56 y/o Type 2

Physician: [Simon Bryant](#)



**Generate reports**  
(Including AGP report\*,  
PDF, print)





**Quick data summary**  
traffic light colour  
coded charts



**A secure environment**  
for your patients to share  
information with you






**Download results from  
CONTOUR® meters**  
without system  
administrator permissions

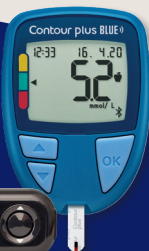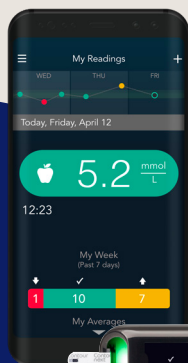

\*Ambulatory glucose profile

Provides understanding of glucose values thanks to:

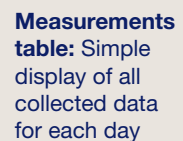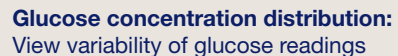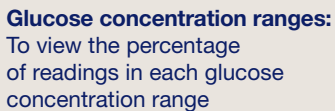

**Results summary:** For a more detailed summary of patient blood glucose (BG)




# Simple onboarding and patients invitation

## Register in 3 simple steps

1. Visit [glucocontro.online](https://glucocontro.online) and click register
2. Fill out information
3. Wait for the approval from the clinic administrator†

### Create an account

#### Account Type


 Patient <input type="radio"/>	 Professional <input type="radio"/>	 Care Giver <input type="radio"/>
Choose this type of an account if you have been diagnosed with diabetes.	Choose this type of an account if you are a healthcare professional.	Choose this type of an account if you want to take care of someone with diabetes but you are not a diabetic.

#### E-mail address


#### First Name

#### Last Name

## Click “Add new Patient”

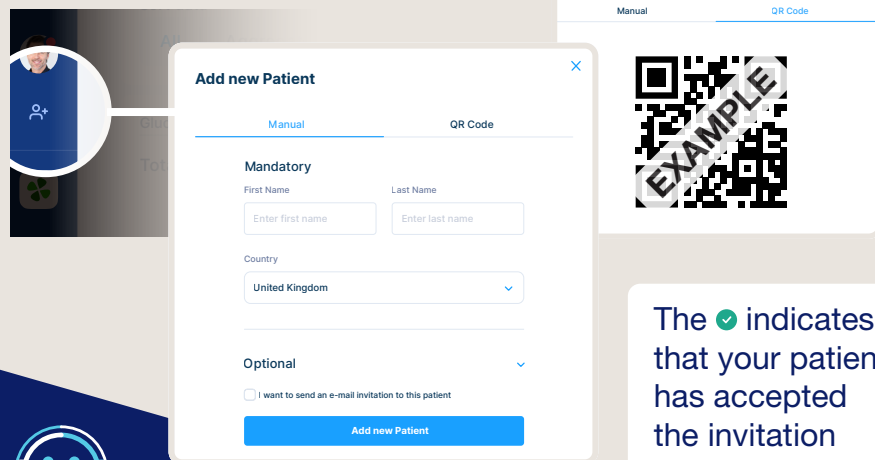

The image shows a composite of three elements: a small circular inset of a person's face, a large 'Add new Patient' form, and a QR code. The form has two tabs: 'Manual' (selected) and 'QR Code'. Under 'Mandatory', there are fields for 'First Name' and 'Last Name', and a 'Country' dropdown menu set to 'United Kingdom'. Under 'Optional', there is a checkbox labeled 'I want to send an e-mail invitation to this patient' which is currently unchecked. A blue 'Add new Patient' button is at the bottom. The QR code is labeled 'EXAMPLE' and is part of a 'Manual' section with a 'QR Code' tab.

The  indicates that your patient has accepted the invitation



**Ascensia Diabetes Support:**

0345 600 6030

[diabetessupport@ascensia.com](mailto:diabetessupport@ascensia.com)

[www.diabetes.ascensia.co.uk](https://www.diabetes.ascensia.co.uk)

† It may take up to 24 hours

© 2021 Ascensia Diabetes Care Holdings AG. All rights reserved.

Ascensia Diabetes Care UK Ltd. Ascensia House, Albert Road, Newbury, Berkshire RG14 1DL.

Ascensia, the Ascensia Diabetes Care logo and Contour are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. All other trademarks are properties of their respective owners and are used solely for informative purposes. No relationship or endorsement should be inferred or implied.

Date of preparation: May 2021. HCP-UK-GluCo092021