UKR (UK Responsible Person): Imported and Distributed by: Ascensia Diabetes Care UK Ltd. Phamaco Distribution (Pty) Ltd. 91 Protea Road Chisinau House Sandton, 2196 Johannesburg, South Africa +27 11 784 0077 www.diabetes.ascensia.com

Imported and Distributed by: Ascensia Diabetes Care Ireland Ltd. 10 Earlsfort Terrace Dublin 2 D02 T380 Ireland Ascensia Diabetes Support 1 800 920 111 - IE

Ascensia Diabetes Care Holdings AG Peter Merian-Strasse 90 4052 Basel, Switzerland EC REP Ascensia Diabetes Care Italy S.r.l. Via Varesina, 162 20156 Milano, Italy

ASCENSIA Diabetes Care

90012925 Rev. 05/24

Microlet® next Lancing Device EN

INSTRUCTIONAL INSERT

- Lancet ejector for safe lancet disposal.
- Black/white adjustable endcap for fingertip testing only.
- Clear AST endcap for alternative site testing only.

Use with MICROLET® lancets for fingertip and palm testing.

WARNING

- Ask your health care professional if Alternative Site Testing (AST) is right for you.
- Do not calibrate a continuous glucose monitoring device from an AST result.
- Do not calculate an insulin dose based on an AST result.

WARNING: Potential Biohazard

- The MICROLET NEXT lancing device is intended for self-testing by a single patient. It must not be used on more than one person due to the risk of infection.
- Do not reuse lancets. Used lancets are not sterile. Use a new lancet each time you test.

INTENDED USE: Intended for collecting capillary blood, for use together with disposable lancets. The **MICROLET NEXT lancing device** is used to draw capillary blood from the fingertip or palm for blood glucose testing with compatible lancets. The **MICROLET NEXT lancing device** is for single-patient use. The lancing device is not intended for neonates. The **MICROLET lancet** is a single-use sterile device intended for drawing a capillary blood sample from an adult or child for glucose monitoring. The lancet is not intended for neonatal use. There are no known contraindications associated with these devices.

CAUTION:

- Contact Customer Service for assistance if your device malfunctions for any reason or if you notice any difficulties using the device.
- Choose alternate puncture sites to prevent excess scarring, which could lead to loss of sensitivity.
- Test your blood glucose level as frequently as recommended by your healthcare provider.*
- Healthcare agencies recommend that adult patients test their BGL three to four times per day (2018 U Oja et al., "Disruption in the diabetic device care market") and that children and young persons test at least five times per day (2016 NICE [NG18], "Diabetes (type 1 and 2) in children and young people: diagnosis and management").

*May not be included in all kits. Available on request from Customer Service.

NOTES: The Capillary Sampling guidelines from the World Health Organization (WHO) recommend a depth for pediatric finger-prick sampling using a lancing device / lancet as follows:

- For a child over 6 months to 8 years: **MICROLET NEXT Setting 1 or 2**
- For a child over 8 years: **MICROLET NEXT Setting 4**

Settings should be adjusted based on comfort. Pressure applied can affect depth. (**WHO Guidelines on Drawing Blood: Best Practices in Phlebotomy**, Geneva: World Health Organization; 2010.)

Get the Blood Drop – Fingertip

FINGERTIP TESTING

Use the black/white adjustable endcap. Do not use the clear AST endcap.

- Attach the black/white adjustable endcap.
- Lock the endcap. Twist the white band until you hear or feel a click and the white and black raised marks line up as shown.
- Insert the lancet firmly into the lancing device. Push until the lancet comes to a complete stop.
- Wash hands and the puncture site with soap and warm water. Rinse and dry well.
- Carefully twist off the round protective cap. Save the cap for disposing of the used lancet.
- Rotate the endcap dial to adjust the puncture depth. The amount of pressure applied to the puncture site also affects puncture depth.

NOTE: Ensure you hear or feel the "click".

If the device does not puncture, re-set by repeating Step 4.

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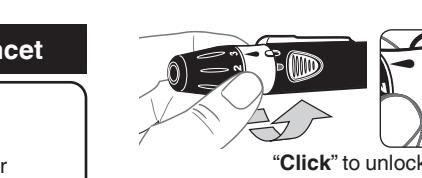
6. Stroke your hand and finger toward the puncture site to form a drop of blood. Do not squeeze around the puncture site.

Eject / Dispose of the Lancet


WARNING
Always dispose of the used lancet as medical waste to prevent injury or contamination to others. Use a new lancet each time you test. **Do not reuse lancets.**

7. Test immediately after you have formed a small, round blood drop, as shown.

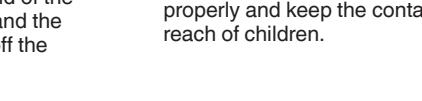
8. Wipe the puncture site with a clean, dry tissue and **maintain pressure** on the site until the bleeding stops.

1. Carefully remove the endcap from the lancet device. Twist the white band of the endcap so the white raised mark and the black indented mark line up. Pull off the endcap.

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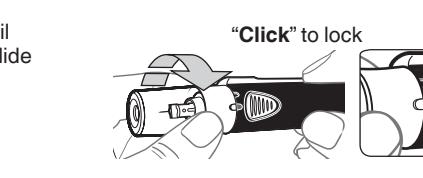CAUTION: Do not use the clear AST endcap on your fingertips.

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2. Attach and lock the **clear AST endcap**. Twist the white band until you hear or feel a click and the white and black raised marks line up as shown.

3. Point the device toward a container for sharp objects (an empty plastic bottle with a lid, for example). Label the container properly and keep the container out of the reach of children.

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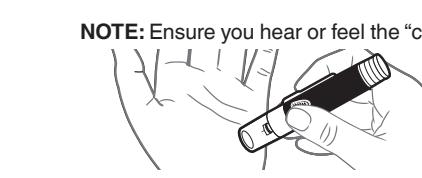
4. Push the grey lancet ejector forward until the lancet falls into the container, then slide ejector back.

NOTE: Ensure you hear or feel the "click."

If the device does not puncture, reset by repeating Step 4.

5. Select a puncture site from a fleshy area on the palm. Avoid veins, moles, bones, and tendons.

6. Do not take the device away from your palm. Maintain steady pressure until a small, round blood drop forms under the clear endcap.

7. Lift the lancet device straight up and away from the skin without smearing the blood drop.

See *Eject / Dispose of the Lancet*.

Get the Blood Drop – Palm

ALTERNATIVE SITE TESTING (AST) PALM

NOTE: Use the **clear AST endcap**.

1. Insert the lancet as described in *Prepare the Lancet Device*.

8. Test immediately.

NOTE: Ensure you hear or feel the "click."

If the device does not puncture, reset by repeating Step 4.

5. Attach and **lock** the endcap. Twist the white band to lock.

6. Pull back the re-set handle to prepare the lancet device. The release button is raised when the lancet device is ready.

7. Lift the lancet device straight up and away from the skin without smearing the blood drop.

See *Eject / Dispose of the Lancet*.

Cleaning and Disinfection

Clean and disinfect your lancing device once a week.

Use cloths dampened with 70% isopropyl alcohol.

CAUTION: Do not use bleach (sodium hypochlorite).

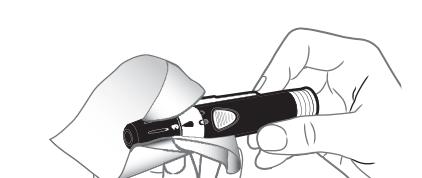
1. Clean the lancing device with 70% isopropyl alcohol and disinfect the lancing device body and endcap (inside and outside). For proper disinfection, you must wipe all surfaces and keep them wet for 2 minutes.

2. Allow to air dry.

3. Remove the endcap and eject the lancet, if present.

4. Get a new cloth dampened with 70% isopropyl alcohol and disinfect the lancing device body and endcap (inside and outside). For proper disinfection, you must wipe all surfaces and keep them wet for 2 minutes.

5. Allow the lancing device body and endcap to air dry.

WARNING
If your lancing device is being operated by a second person who is providing testing assistance to you, the second person should wear gloves and the lancing device should be disinfected prior to use to reduce the risk of transmitting blood-borne pathogens.

SIGNS OF DETERIORATION: Your lancing device has been tested for 260 cycles of the cleaning and disinfection procedure (one cycle per week for 5 years). The lancing device has been demonstrated to withstand 5 years of cleaning and disinfection without damage. However, if your device malfunctions for any reason, contact Customer Service. See *Customer Service*.

Customer Service

OTY = Quantity

Keep dry

UDI = Unique device identifier

Single sterile barrier system with protective packaging outside

Do not use if package is damaged

Low-density polyethylene*

Non-corrugated fiberboard (paperboard)*

PAP = Paper*

*Separate waste collection. Check your local municipal guidelines.

MICROLET NEXT lancing device made in Poland. Ascensia, the Ascensia Diabetes Care logo, and Microlet are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG. All other trademarks are properties of their respective owners. No relationship or endorsement should be inferred or implied. © 2023 Ascensia Diabetes Care Holdings AG. All rights reserved. www.diabetes.ascensia.com

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© 2023 Ascensia Diabetes Care Holdings AG. Alle rechten voorbehouden.